

- **How to integrate emergency stop?**

The emergency stop can be integrated into the charging ports of your powered wheelchair. The charging ports are available in the joystick and in the Omni2 module.

There are two types of emergency stops. The first type is the traditional emergency stop button, while the second type is the remote emergency stop.

In the case of the traditional emergency stop button, the button is connected to the charging port of your powered wheelchair, and it is operated by pressing on the button.

In the case of the remote-controlled emergency stop, the receiver is connected to the charging port of your powered wheelchair. The transmitter is portable and handheld. By pressing the button on your transmitter, an infrared signal activates the receiver leading to an emergency stop.

Caution: When operating a remote-controlled emergency stop, you must be within the operational range of the transmitter.

Indicative emergency stop devices: Sunrise Medical Switch IT Remote Stop Switch, Power2Go Safety Stop Wireless Remote Wheelchair Kill Switch Kit, ASL 504 Remote Emergency Stop Switch for Rnet.

