

- **How to allow the attendant to take control when the patient is driving?**

When the patient is driving, the attendant can take control by pressing the “**PROFILE**” button on the attendant joystick. This activates the dedicated attendant Profile.

This is configured in the Rnet Programmer under **Profile Management**, in the dedicated attendant Profile. Set **Allow Grab** to “**Yes**”.

Profile Management	Profile 1	Profile 2	Profile 3	Profile 4	Profile 5	Profile 6	Profile 7	Profile 8
Profile Name	Drive	CoMoveIT Smart L	CoMoveIT Smart H	SC Cruise No Assist	SC Cruise	Profile 6	No assist	Attendant
Profile Enable	Yes	Yes	Yes	No	No	No	No	Yes
Mode Enable	[12345678]	[12345678]	[12345678]	[1 ]	[12 ]	[1 ]	[1 ]	[12 ]
Input Device Type	JSM	Omni	Omni	Omni	Omni	Universal	Universal	Attendant
Input Device Subtype	All	All	All	All	All	All	All	All
Seat Reversal Profile	No	No	No	No	No	No	No	No
Allow Grab	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes

Under **Controls** tab, **Global** subtab, set **Change Profile while Driving** to “**Yes**”.

Controls	
Global	
Momentary Screens Enabled	Yes
Change Profile while Driving	Yes
Change Speed while Driving	Yes
Actuator Switches while Driving	Yes
Speed Adjust	Yes
Profile Button	Profiles
Actuator Endstop Beep	No
Sounder Volume	1
Horn Volume	Low
Start-Up Beep	No
Lock Function Enabled	Sequence
Reverse Driving Alarm	No
OBP Keycode Entry	No
Power-up Mode	Last used
External Profile Jack Detect	Off
External On/Off Jack Detect	Off

**Caution:** When the attendant is driving the powered wheelchair, the user presses the user switch sensor, e-stop is caused.