



- How to control TVs and similar devices such as Radios, DVDs, Blu-ray Players, etc.?

To control TV and similar devices please consult sections “**Control Multiple Peripherals from one Device**” and “**Peripheral Control Using Housemate**”. If none of these methods are suitable for you, then the traditional Infrared communication embedded in the Omni2 Module can be used with CoMoveIT Smart.

The Omni2 Display Section includes an IR transmitter and receiver that allows it to replicate commonly used IR devices, such as remote controls for TVs, DVDs, Cable/Satellite or environmental controls such as automatic door openers, and any other device that uses IR communication.

IR Mode is accessed via the User Menu on the Omni2 screen. Any function of your device’s remote control can be programmed on the IR Control of Omni2.

The IR Control Mode is set up using the Rnet Programmer software. In the Rnet Programmer, under **Configuration** tab, create a Mode for the IR Control. It is recommended to give a distinctive name to the Mode (e.g. **IR Control**). Set **Input** to “Raw” and select the **Output “IR Control”**.

	Profile 1	Profile 2	Profile 3	Profile 4	Profile 5	Profile 6	Profile 7	Profile 8
<b>Profile Management</b>								
Profile Name	CoMoveIT Smart IN	CoMoveIT Smart L	CoMoveIT Smart M	CoMoveIT Smart H	CoMoveIT Smart VH	Profile 6	Profile 7	Attendant
Profile Enable	Yes	Yes	Yes	Yes	Yes	No	No	Yes
Mode Enable	[1234 7]	[1234 7]	[1234 7]	[1234 7]	[1234 7]	[ ]	[ 7]	[1234 78]
Input Device Type	Omni	Omni	Omni	Omni	Omni	Omni	Omni	JSM
Input Device Subtype	All	All	All	All	All	All	All	All
Seat Reversal Profile	No	No	No	No	No	No	No	No
Allow Grab	Yes	Yes	Yes	Yes	Yes	Yes	Yes	Yes
<b>Configuration</b>								
Mode Name	Mode 1	Mode 2	Mode 3	Mode 4	Mode 5	Mode 6	Mode 7	Mode 8
Drive	Drive	Seating	IR Control	Mitch	Mode 5	Mode 6	My Devices	Programming
Input	Drive	Seating	Raw	Drive	Seating	Seating	Raw	Programming
Intermediate	None	None	None	None	None	None	None	None
Output	Drive	Seating	Drive Seating Environmental Programming Mouse 1 Output 3 Output 4 Output 5 Output 6 Output 7 IR Control Mouse 2 iDevice Robotic Arm	None	None	None	None	None
Speeds	CoMoveIT Smart IN	CoMoveIT Smart L						
Controls								
Latched	CoMoveIT Smart IN	CoMoveIT Smart L						
Seating								
Battery								
General								
Motor								
Inhibits								
Lights								

**Caution:** This control method can only be used with devices that support Infrared (IR) control.



+ IR of Omni2